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## ENGLISH Technical notice

(Translation of the original French notice)

### COSMOS / ELECTRIC DEHORNERS

#### IMPORTANT - DO NOT DISCARD

Read this instruction manual carefully in order to familiarize yourself with the device before using it. Keep this manual for future reference.

### INTRODUCTION

COSMOS is a compact dehorner that enables automatic management of the temperature and cycle time.

Device power:

- Cat. No. 124/125 : 90 W

- Cat. No. 126/127 : 80 W

There are battery and mains-powered versions of this device:

#### Battery version

- With 16 mm Ø cattle tip, Cat. No. 124

- With 14 mm Ø goat tip, Cat. No. 125

#### Mains version

- With 16 mm Ø cattle tip, Cat. No. 126

- With 14 mm Ø goat tip, Cat. No. 127

### WARNINGS

- This device is not a toy; nor is it a tool that can be used for purposes other than that for which it was designed. It may only be used for the cauterization of blood vessels supplying horn buds, under the conditions specified in the relevant standards, directives and recommendations. GUILBERT EXPRESS may in no way be held liable for any other use.
- This device may be used by children aged at least 8 years old and by persons with physical, mental or sensory impairments, or lacking in experience or knowledge, provided that they are properly supervised, or have received instructions on how to use the device in total safety, and have understood the risks involved. Children must not play with this device. User cleaning and maintenance of this device must not be performed by children without supervision.
- Do not immerse this device in water.
- Do not place this device in fire or on a hot surface (> 40 °C / 104 °F).
- Do not place the tip of this device against skin, even when charging.
- Do not place this device in the mouth, even when charging.

- Do not hold this device by the tip, even when charging.
- Non-compliant, inappropriate or improper use can cause serious burns. (Temperature at the tip > 700 °C / 1292 °F).
- This device is designed to perform one cycle per minute.
- Non-compliance with the instructions annuls the guarantee.
- Do not discard this device in a rubbish bin. It contains components that must be disposed of in accordance with applicable regulations. Regarding the battery-version of the COSMOS device, the battery must be removed before the dehommer is disposed of. In order to do this, open the device by removing the 7 screws, cut the wires connected to the battery and then remove the battery. The battery must be disposed of in accordance with applicable standards in your country.

In order to ensure the safety of users, and the useful life and optimal operation of the equipment, the following rules must be followed:

- Keep the device well away from all heat sources.
- The device must be protected from knocks.
- Do not melt anything on the heating tip. Before use, check that the tip is not obstructed by any objects.
- Never press the heating part against a metal or inflammable object.
- Following use, even when the power is off, the heating tip remains hot and can cause serious burns.
- Do not use the device if the heating wire has been damaged, in particular if it has been crushed or cut.
- Only replace spare parts using original spare parts.
- Never open or modify the device.

#### SPECIFIC FEATURES OF THE BATTERY-VERSION COSMOS

- The device must be protected from humidity and any risk of being splashed by liquids.
- Before use, recharge the device by following instructions in the section «Use / Charging the battery».
- The battery must be stored and charged in an environment with a temperature of < 10 °C / 50 °F and < 25 °C / 77 °F.
- The battery must be stored and charged in an environment that is dry and well away from any inflammable materials.
- When connecting to the power supply, ensure that the device and socket outlet are dry.
- Never allow the battery to fully discharge. In the event of extended storage without use, recharge the battery at least once every 6 months.
- Never use the device when it is connected to the power supply

#### SPECIFIC FEATURES OF THE MAINS-VERSION COSMOS

- We recommend protecting the device from rain and humidity. There is a risk of electric shock if water enters this electrical device.
- Never use the mains power cable to carry or hang this device, or to unplug it from the socket outlet. Keep the mains power cable away from heat, oil and cutting edges. Do not touch the mains power cable if it is damaged or severed during use. Immediately unplug it. A damaged or twisted mains power cable can cause electric shocks and must be replaced by the manufacturer, the manufacturer's after-sales service, or a person with a similar qualification, in order to prevent any risk of danger.
- This device must only be used under shelter.
- In the event of a break lasting more than 30 minutes, unplug the device from the socket outlet.
- Allow the device to fully cool down before storing it.
- The mains power cable must always be kept to the rear of the device.

## SYMBOL DEFINITIONS

- Read the user manual.

- Removable power supply unit.

- Must be recycled in accordance with applicable standards in the local country.

- Compliant with the European standard.

- Caution: overheating device. Allow it to cool down.

- Device with class-2 electrical insulation.

- Device with class-3 electrical insulation.

## USE

### Charging the battery (Cat. No. 124 and 125 only)

Before using for the first time, it is essential to continuously charge the device in order to optimize the battery lifetime. The battery will reach full capacity following several charging/discharging cycles.

#### Charging the battery:

- Remain in a cool, dry place, well away from any inflammable products.
- Turn the switch to the «ON» position (Fig. 1-A).
- Plug in the charger cable (Fig. 2).

CAUTION: The battery must only be charged using the charger supplied with the device, with the Cat. No. 12003.

- Switch on the power supply to the charger, the indicator lights (Fig. 1-B) will blink.
- Once charged, the indicator lights (Fig. 1-B) turn green. Unplug the charger and turn the switch back to the «OFF» position (Fig. 3).

## USAGE RULES AND MALFUNCTIONS

### BATTERY-VERSION COSMOS CAT. NO. 124 AND 125

To switch the device on, turn the switch (Fig. 1-A) to the «ON» position.

#### The indicator lights (Fig. 1-B) have two functions:

They indicate progress of the cycle. They turn off when the dehorning cycle is complete.

They indicate the battery charge level:

- 1 light - the level is 0 to 20%.
- 2 lights - the level is 20 to 70%.
- 3 lights - the level is 70 to 100%.

### MAINS-VERSION COSMOS Cat. No. 126 and 127

Once it is plugged in, a slight heating of the device occurs. The feeling of heat will increase during use. This is normal. Always unplug your device once your dehorning session is finished.

In the event that the device overheats, the indicator light (Fig. 1-B) turns red and the cycle will be unable to continue. Wait until the indicator light turns green again.

### The indicator light (Fig. 1-B) indicates the following:

- «Green»: the device is powered.
- «Red»: the device is overheating. Wait for it to turn off.

Malfunction	Causes and solutions
There is no signal tone and the indicator light does not turn on when the trigger is pressed (Fig. 6).	<b>Cat. No. 124 and 125</b> <ul style="list-style-type: none"><li>• Your battery is fully discharged. Charge your battery.</li><li>• You have not unplugged your charger. The heating wire has not been properly tightened (Fig. 4).</li></ul> <b>Cat. No. 126 and 127</b> <ul style="list-style-type: none"><li>• You have not plugged your device in or the socket outlet is defective.</li><li>• The heating wire has not been properly tightened (Fig. 4).</li></ul>
The charger is plugged in, the indicator light (Fig. 1-B) does not turn on.	Either your charger is not properly plugged in or it is defective.

## CAUTERIZATION OPERATION

Before performing the operation, check that the applicable rules have been followed. In order to ensure that the operation is properly performed, we recommend that you follow the rules set out in the «Dehorning recommendations» paragraph.

### Operating method

- 1 - Position the heating element (Fig. 5) on the base of the horn bud.
- 2 - Press on the trigger (Fig. 6). Hold the trigger down throughout the operation.
  - Cat. No. 124 and 125: the indicator light (fig. 1-B) turns on, indicating that the cycle has begun.
  - Cat. No. 126 and 127: a steady tone indicates that the cycle is underway.
- 3 - End of operation.
  - Cat. No. 124 and 125: the indicator light (fig. 1-B) switches off and a steady tone signals that the operation is complete.
  - Cat. No. 126 and 127: the steady tone ends, indicating that the cycle is complete.
- 4 - Release the trigger (Fig. 6).

## ROUTINE MAINTENANCE

### Never attempt to modify this tool

Important: Never use a liquid to clean the device. No specific maintenance operations are required for this device.

### Battery version

In order to preserve battery life, the following rules must be followed:

- Never store the device with the battery discharged.
- In the event of extended storage without use, recharge the battery once every 6 months.
- Store the device in an environment with a temperature of between 10 °C and 20 °C / 50 °F - 68 °F.

### Cleaning the tip

It is recommended to remove accumulated dirt in the tip (Fig. 5). To do this:

- 1 - Press on the trigger (Fig. 6).
- 2 - Wait for 12 seconds.
- 3 - Release the trigger (Fig. 6).
- 4 - Once the tip has cooled down, use a brush to remove residue.

If the heating wire (Fig. 7-A) and/or the ceramic bit (Fig. 7-B) are damaged, they must be changed.

To do this, perform the following steps with the device unplugged:

- Remove the 2 screws (Fig. 7-C) using the key provided (Fig. 7-D).

- Remove the heating wire (Fig. 7-A).
- Remove the ceramic bit (Fig. 7-B).
- Replace with another ceramic bit and heating wire (Fig. 7).

## SPARE PARTS

Tip formed of a ceramic bit and heating wire:

Cattle tip internal Ø 16 mm

Ceramic bit Cat. No. 12001 (Fig. 8)

Heating wire Cat. No. 12016 (Fig. 9)

Goat tip internal Ø 14 mm

Ceramic bit Cat. No. 12002 (Fig. 10)

Heating wire Cat. No. 12014 (Fig. 11)

Chargers: Cat. No. 12003 (Fig. 12)

## AFTER-SALES SERVICE

This tool is designed exclusively for professional applications and must only be used by users who have received appropriate training. For reasons of safety, repairs must only be performed by the manufacturer. The guarantee shall not apply in the event of non-compliance with this clause.

Always protect the tool from extreme stresses and impacts (flames, passage of vehicles, etc.)

Wipe the outside of the tool with a damp cloth in order to keep it clean. For any other problems, contact your EXPRESS retailer. GUILBERT EXPRESS may in no way be held liable in the event that this product is not used in strict compliance with the above instructions.

GUILBERT EXPRESS is not liable for any injury, loss or damage, either direct or consequential, caused by the use of or the inability to use this product, nor for any environmental complaints. It is understood that GUILBERT EXPRESS has no means of controlling the end use of its products. Consequently, it is the responsibility of the buyer to determine whether or not the products are suitable for the intended use and the buyer assumes all risks and responsibilities related to the products.

## DEHORNING RECOMMENDATIONS

Performing this operation on young animals makes it easier to restrain and dehorn them and limits stress caused to the animals. It is therefore recommended to perform dehorning before the age of 4 weeks for calves and before 2 weeks for kids. Check with your veterinarian.

The idea of this device is to prevent horn growth by cauterizing blood vessels.

### 1. Immobilizing the calf or kid

Proper restraints should be applied. A moving animal will make it impossible to perform even cauterization around the horn bud. A general and/or local anaesthetic may be used, in accordance with legislation in your country.

### 2. Shave around the horn bud before performing cauterization in order to:

- properly visualize the location of the horn bud,
- avoid getting fur in the wound, which can be a source of infection,
- properly visualise the quality of the cauterization.

### 3. Do not attempt to remove the horn bud

An even ring of cauterization will prevent horn bud growth.

### 4. Inspect the cauterization ring

Properly inspect the quality of this ring (it should be 4 to 5 mm deep, detached all the way around, and the white membrane known as the periosteum should be visible). Partial regrowth of horns is often the result of imperfect work:

- 1 - Dehorning was performed too late. Make sure that you dehorn animals sufficiently early.
- 2 - The diameter of the dehorner tip was unsuitable and did not destroy all of the surrounding tissue.

Check that your tip is suitable for the type of dehorning planned. Change the tip if required.

- 3 - Not all of the veins were cauterized. It is therefore essential to check that the ring of cauterization around the horn bud

is continuous and unbroken. This is essential for successful heat dehorning. In order to check, perform a continuous circle around the horn bud, at an angle of 45°, to complete the cauterization. This will ensure that blood vessels are totally destroyed.

### 5. Disinfect

As soon as the operation is complete, apply a disinfectant spray.

This spray will have two effects:

- cooling of the cauterized area,
- prevention of subsequent infection.

Make sure that the spray penetrates all the way inside the wound.

### Our recommendations:

Perform dehorning before the 29<sup>th</sup> day following the animal's birth (ideally before the 15<sup>th</sup> day).

Use a fully effective restraint system, such as a restraint cage.

Shaving around the horn bud will enable you to properly view and inspect your work.

Cat. No. 124 and 125: Do not stop the operation until you hear the tone at the end.

Cat. No. 126 and 127: Do not stop the operation until the steady tone ends.

Use a spray to protect and disinfect the wound.

Make sure that no flies lay eggs in the wound. Use fly repellent if necessary.

Check that there is no bleeding and that your animal is feeding properly in the hours following the operation.

For effective pain management, we recommend the use of local anaesthetic and an anti-inflammatory.

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